

## Quick Start Instructions

### Step 1

#### INSERT BATTERIES (Not Included)

- Insert 4 "C" batteries into the 30-Channel Receiver
- Insert 2 "AA" batteries into the Caregiver pager.
- Nurse Call Buttons (batteries included).




### Step 2

#### PROGRAM NURSE CALL BUTTONS

- Hold down "Record/Add" button located on the receiver. Press and Hold until receiver beeps 3 times.
- Once the receiver beeps 3 times, press the desired Nurse Call Button (if programming a 30A-MS motion sensor, turn on motion sensor; if programming 30A-NCA monitor adaptor, plug 30A-NCA into desired monitor, then press and release pressure pad or step on floor mat; if programming TM-02 GhostCord® Transmitter, plug TM-02 into desired cordless pressure pad or floor mat, then press and release pressure pad or step on floor mat) until the receiver beeps once and displays a number. Number shown is that Nurse Call Button's identification number on the receiver.

**Please note that the monitor will be shipped with the alarm volume set at 90 decibels (90 decibels is the legally accepted occupational noise allowance).**

- Wait for one beep (it takes a few seconds) and/or the current number being displayed stops flashing, then press the next Nurse Call Button to be programmed. Continue this process until all desired Nurse Call Buttons are programmed. Once 30 Nurse Call Buttons, are programmed, additional buttons will added to existing channels.
- Receiver will beep before it exits programming mode and all numbers on the screen will disappear.
- To prevent others from erasing programming, flip the switch on the side of the 30A-CMU Central Monitor to the lock position.
- Make sure the switch on side is positioned to "Call" and not "Alarm".

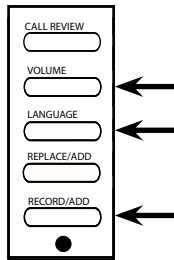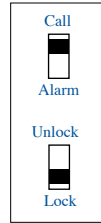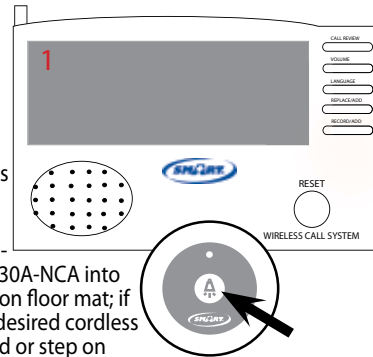

### Step 3


#### ADJUST LANGUAGE AND VOLUME

- To adjust language or to select a song, press the "language" button on the 30A-CMU Central Monitor and scroll through to find the desired language or song.
- Adjust volume by pressing the "Volume" button on the 30-Channel Receiver.

### Step 4

#### TEST SYSTEM

- Test system by pressing the Nurse Call Button. Pager and Receiver should alarm.
- NOTE: Turn pager on by flipping on/off switch on side of pager. When on, the pager will respond to any 30A-CMU signal within 200 ft. Lock settings on pager by flipping switch located inside battery compartment. Pagers read 1-30.
- When the alarm is activated press the "Reset" button on the
- Receiver to acknowledge activation and to reset the Nurse Call Button.
- Hang Nurse Call Buttons on the wall using double sided tape or wear it as a pendant around the neck.
- Attach Pager to caregiver, or place pager where it can be heard by the caregiver.
- **IMPORTANT:** End user is responsible for testing system at it's furthest range of use. This is to ensure that the caregiver understands the range of the system and leaving it will make the system ineffective.

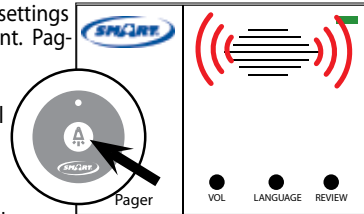

#### Important Information:

30-Channel Receiver, Nurse Call Button's and Pagers rely on wireless technology which is subject to physical and environmental considerations. These products do not have an out of range function and as such should be tested periodically in the setting in which they are to be used to understand their area of effective operation. Please be aware that the transmitter will not be 100% accurate if it is out of range at anytime. Wireless systems and monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use.

**Warning:** Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. Test this device before each use. Read the instructions and Legal Disclaimer.

Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.

The Smart Caregiver Corporation 30-Channel Wireless Call System & Nurse Call Paging System are designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including, but not limited to:

- Direct patient supervision
- Adequate training for staff personnel for elopement
- Testing of the system before each use.

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

**WARNING: The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.**

**Need Set-up Help?**

**Call Our Friendly Customer Service Reps at (800) 650-3637**



1229 N. McDowell Blvd., Petaluma, CA. 94954  
 sales@smartcaregivercorp.com  
 www.smartcaregivercorp.com  
 Toll Free: (800) 650-3637




# 30A-CMU Wireless Central Monitor & Nurse Call Paging System

## Installation Instructions

1229 N. McDowell Blvd., Petaluma, CA. 94954  
sales@smartcaringivercorp.com  
www.smartcaringivercorp.com  
Toll Free: (800) 650-3637



## Installation Instructions

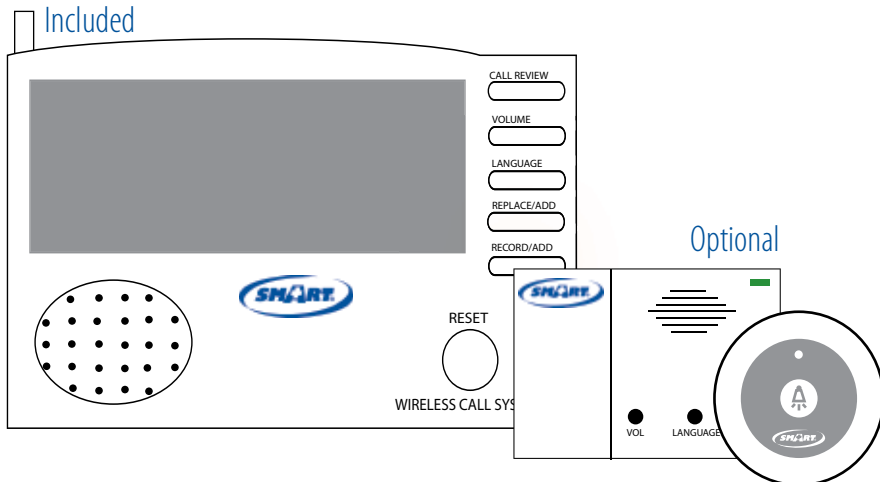

30A-CMU Wireless Central Monitor & Nurse Call Paging System

## 30A-CMU Wireless Central Monitor & Nurse Call Paging System

This 30A-CMU Central Monitor is programmable for up to 60 residents (use 30A-NC). The 30A-CMU Central Monitor identifies the resident who called (via the Nurse Call Button/Pendant 30A-NC), while the Caregiver is notified on the Caregiver Pager (30-PG). System must be located within 300 feet of the button/pendant. This Unit comes with a 60 Day Warranty. Pagers and Nurse Call Buttons sold separately.

### 30A-CMU Contents:

- 30A-CMU Central Monitor
- 2 Screws
- 1 Mounting Bracket

### 30A-CMU Optional Accessories (sold separately):

- Caregiver Pager without LCD Display (30A-PG)
- Caregiver Pager with LCD Display (30A-PGD)
- Nurse Call Button (30A-NC)
- Wireless Motion Sensor (30A-MS)
- Wireless Bed & Chair Fall Alarms (Call for details)
- Exit Contact Switch for Door or Window (30A-EXT)
- AC Adapter (AC-04)