


# Pull String Monitor Set-Up Instructions


## Chair Use —

1. Insert 9V battery in monitor (battery is not included).
2. Attach alarm to arm of chair/wheelchair using the strap provided or to the back of chair/wheelchair using the bracket provided. Make certain alarm is securely attached and out of resident's reach.
3. Attach string to patient's clothing with clip (on end of string).
4. Adjust string length using the cord stop so alarm engages when resident attempts to get out of the chair.

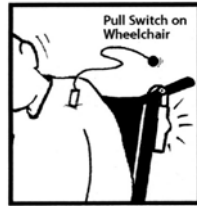

## Bed Use —

1. Insert 9V battery in monitor (battery is not included).
2. Attach alarm to back of headboard using the bracket provided. Make certain alarm is securely attached and out of resident's reach.
3. Drape string over headboard & attach string to patient's clothing with clip (on end of string).
4. Adjust string length using the cord stop so alarm engages when resident attempts to get out of the bed.

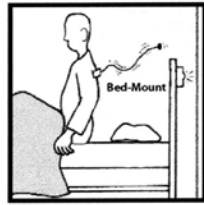

## Alarm Volume

Please note that the monitor will be shipped with the alarm volume set at 90 decibels (90 decibels is the legally accepted occupational noise allowance).

## How to Adjust String Length

To adjust cord length, depress button on cord stop, move cord stop to desired length. The appropriate length is determined by caregiver and should allow resident freedom of movement. The cord length will engage alarm in the event of the resident attempting to get out of their chair/bed. Make sure cord is not obstructed by any objects, such as pillows or clothing.

PROPER LENGTH OF CORD IS RESPONSIBILITY OF CAREGIVER.  
TEST THIS AND ALL ALARMS BEFORE EACH USE.

**WARNING: The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.**

**WARNING: The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.**

**Warning: Failure to comply with warning may result in injury or death.** This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will prevent falls without fail. Test this device before each use. Read the instructions and Legal Disclaimer.

Seller warrants that the goods sold will be free from defect of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, **NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.**

The Smart Caregiver Corporation Fall Prevention System is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including, but not limited to:

- a.) Direct patient supervision.
- b.) Adequate training for staff personnel for elopement.
- c.) Testing of the system before each use.

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF ANY USE THE GOODS.

All **Smart Alert Systems** are intended to augment, not replace fall prevention measures that caregivers have in place, or the caregiver's vigilance. **Smart Alert Systems** should be used in conjunction with a comprehensive fall prevention program.

**Smart Alert Systems** are not a substitute for physical restraints and cannot be used in lieu of restraints in cases involving residents whose conditions necessitate the use of physical restraints.

**Smart Alert Systems** should be used in conjunction with restraints as part of a complete safety and fall prevention program.

**Smart Alert Systems** will not deter, but not restrict a resident from getting up, out of chair or bed.

**Smart Alert Systems** do not act as a restraint in any way.

**Smart Alert Systems** rely entirely on an immediate response from the caregiver to the resident upon activation of the alarm and is not a substitute for the visual monitoring of residents that are a high risk for fall.

**Smart Alert System** can be defeated by a person with lucidity, or a cognitively aware person, therefore, the system should be used by caregivers judiciously. The caregiver must assume active patient support at all times.

Keep the **Smart Alert System** in good working condition and operational. Battery maintenance must be done by the caregivers, or a return from **Smart Caregiver Corporation** for repairs.

Only mount the alarm as these instructions indicate. If you have any questions about proper placement, usage, or operation of any part of the **Smart Alert Systems**, consult with a technician before you attempt to use the system.

**Smart Alert Systems must be tested every time before use. It is caregiver's responsibility to make sure that the alarm is operational and the cord is adjusted to the right length.**



1229 N. McDowell Blvd., Petaluma, CA 94954  
Toll Free: 800.650.3637 • Fax: 707.781.7440  
email: sales@smartcaregivercorp.com  
www.smartcaregivercorp.com