


Installation Instructions

TL-2000 "Unbreakable" Pull-String Monitor

Bed Use — Pull-Switch & Magnet Switch monitors

1. Attach alarm on wall above headboard, or on back of headboard, using the Velcro provided. Make certain alarm is securely attached and out of resident's reach.
2. Connect 9v battery in monitor (battery is not included).
3. Drape cord over headboard & attach cord to patient's clothing with clip.
4. Adjust cord length so alarm engages when resident attempts to get out of bed.

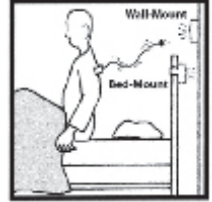

Chair Use — Pull-Switch & Magnet Switch monitors

1. Attach alarm to arm or crossbar of chair/wheelchair using the strap provided. Make certain alarm is securely attached and out of resident's reach.
2. Connect 9v battery in monitor (battery is not included).
3. Attach cord to patient's clothing with clip.
4. Adjust cord length so alarm engages when resident attempts to get out of chair.



How to Adjust Cord Length

To adjust cord length, depress button on cord stop, move cord stop to desired length. The appropriate length is determined by caregiver and should allow resident freedom of movement. The cord length will engage alarm in the event of the resident attempting to get out of their chair/bed. Make sure cord is not obstructed by any objects, such as pillows or clothing.

PROPER LENGTH OF CORD IS RESPONSIBILITY OF CAREGIVER.

Alarm Volume

Please note that the monitor will be shipped with the alarm volume set at 90 decibels (90 decibels is the legally accepted occupational noise allowance).

TEST THIS AND ALL ALARMS BEFORE EACH USE.



Toll Free: 800.650.3637 • Fax: 707.781.7440
email: sales@smartcaregivercorp.com
www.smartcaregivercorp.com
1229 N. McDowell Blvd., Petaluma, CA 94954

Warning: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. Test this device before each use. Read the instructions and Legal Disclaimer.

Seller warrants that the goods sold will be free from defect of workmanship and manufacturing , for a period of one year from the date of sale. Other than this limited warranty, seller makes no express of implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.

The Smart Caregiver Corporation Fall Prevention system is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including, but not limited to:

- a.) Direct patient supervision.
- b.) Adequate training for staff personnel for elopement.
- c.) Testing of the system before each use.

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF ANY USE THE GOODS.

WARNING: The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.