

Installation and Use Instructions

TL-3005SYSR1 DepartAlert™ Single Door Anti-Wandering System

- Simple Installation
- No Hard Wiring



SMART CAREGIVER CORPORATION



TL-3005SYSR1 DepartAlert™ Anti-Wandering System

IMPORTANT INFORMATION: DepartAlert™ Anti-Wandering System relies on wireless technology which is subject to physical and environmental considerations. These products do not have an out of range function and as such should be tested periodically in the setting in which they are to be used to understand their area of effective operation. Please be aware that the transmitter will not be 100% accurate if it is out of range at anytime. It is the end-user's responsibility to make sure that the product is used correctly and within range of each other. Wireless systems and monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use. It is important to read and follow the directions.

WARNING: Failure to comply with the warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by a caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. **Test this device before each use.** [Read the instructions and Legal Disclaimer.](#)

The Smart Caregiver Corporation TL-3005SYSR1 is designed to be installed by the end user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

A.) Direct patient supervision B.) Adequate training for staff personnel for elopement


C.) Testing of the system before each use

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.

Limited Warranty: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.

TL-3005SYSR1 DepartAlert™ Single Door Anti-Wandering System

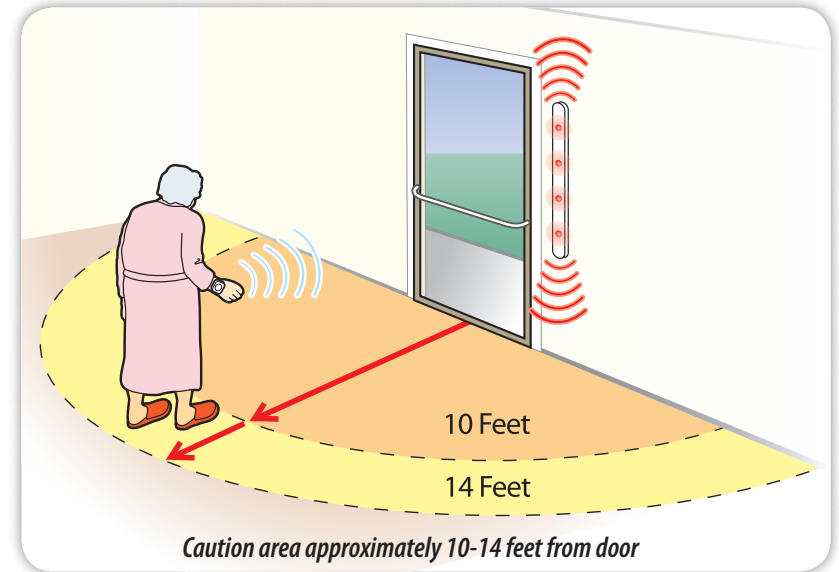

About The DepartAlert™ Anti-Wandering System:

The DepartAlert™ Anti-Wandering System projects a sensor field and alarms visually and audibly when a resident wearing a wristband transmitter passes within the sensor field's range. This system can be configured for use with a Central Monitoring Unit that will identify specific residents and or doorways triggered the alert and works up to 999 residents. DepartAlert™ can also be configured for double doors.

TL-3005SYSR1 CONTENTS:

- 1 Door Alarm Bar (TL-3005)
- 1 Resident Wristband Transmitter (TL-2012SR1)
- 1 Caregiver Key (CK-01D)
- 1 AC Power Adapter (AC-03)
- 1 Magnetic Contact Switch (TL-2024)
- 2 Double Sided Tape Strips

TL-3005SYSR1 OPTIONAL ACCESSORIES

(sold separately):

- Wall Mount Reset Button (TL-2014W)
- Wireless Key Pad (TL-2010WKP)
- Door Alarm Reset/Tester & Wristband Tester (DT-01)
- Central Monitoring Unit (TL-2015R2)
- Magnetic Door Lock (TL-2025)
- Emergency Call Alarm Light (2007-ECA-REV1)

To meet industry standards, door bars and wristbands should be tested at regular intervals to assure proper functionality. A standard procedure should be initiated at the facility for this testing. It is recommended that a tester be purchased for all facilities using SMART Anti-Wandering Door Systems.



SMART CAREGIVER CORPORATION

1229 North McDowell Blvd., Petaluma, CA. 94954 | Tel: 800-650-3637
sales@smartcaregivercorp.com | www.smartcaregivercorp.com


Need Set-up Help? (800) 650-3637

1 Mount Door Monitor to the Wall

A Peel off yellow backing from adhesives (3) on the back of the door monitor. The door bar should be mounted to clean, smooth surfaces




B Adhere the door bar vertically centered to the outside of either door frame with cords facing up or down (in direction of power source).

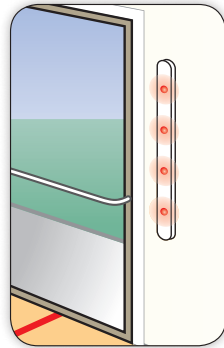

Warning: If mounting door monitor near metal frame, mount at least 8" away from metal frame.

2 Plug in the AC Adapter

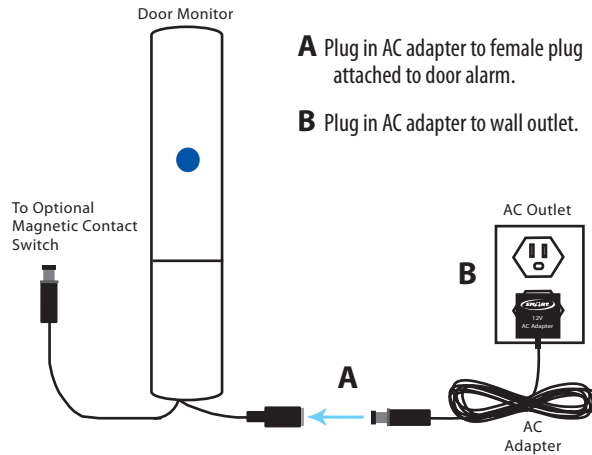

A Plug in AC adapter to female plug attached to door alarm.


B Plug in AC adapter to wall outlet.

Note: Alarm will illuminate and beep to indicate system is functioning.

3 Activate the Resident Transmitter(s)

A Place orange magnet on the face of the white transmitter in the lower right hand corner.

B Transmitter will blink 3 times to indicate transmitter has been activated.

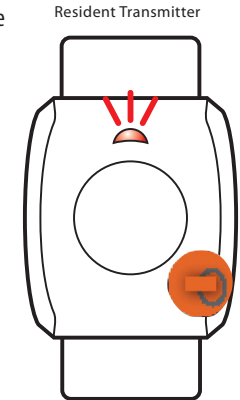

Warning: Transmitter bands are not to be placed on a wheelchair, walker or near jewelry. The correct placement is the the resident's wrist.

4 Test the System

A Test the active range of the door monitor by approaching the door with a resident transmitter activated from all different directions, making sure that alarm goes off from desired distances.


B To silence alarm, place the orange magnet over the blue sticker on the door monitor.

Orange Magnet



C To retest, wait approx. 20 seconds for monitor to blink (single light) before the system is ready.

D Attach transmitter to resident's wrist with included strap.

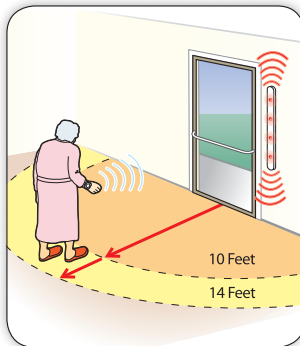


Warning: Door monitor will sound if door is opened and a resident wearing a transmitter is within 10-14 feet from door monitor. If the alert is triggered when a transmitter is too close or too far from door, or if you can pass thru door, call for technical assistance.

5 (Optional) Mount Magnetic Contact Switch

WHEN USING THE MAGNETIC SWITCH, THE ALARM WILL ONLY SOUND WHEN DOOR IS OPENED AND IF A RESIDENT WEARING A WRISTBAND IS WITHIN THE MONITORED FIELD.

A Screw magnetic contact (see below) to underside of door frame with wording facing open door. Lead cord along outside of door frame.

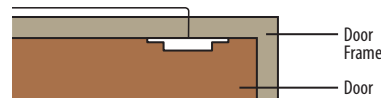

B Screw magnet (see below) to door with arrow facing up.



C Magnetic contact and magnet must align perfectly to function.

D Plug in magnetic contact switch to male plug attached to door alarm.

6 Test the System with Magnetic Contact Switch

A Test the door monitor with the magnetic contact switch installed by opening the door with an activated resident transmitter in hand. Door monitor will sound when door is opened.


B To silence alarm, place the orange magnet over the blue sticker on the door monitor.

Orange Magnet



C To retest, wait approx. 20 seconds for monitor to blink (single light) before system is ready.

D Attach the transmitter to the resident's wrist with the included strap.

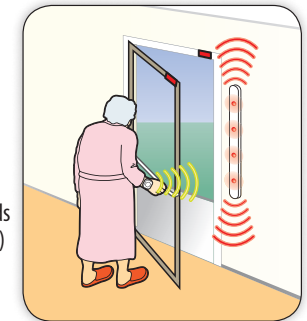

Warning: Door monitor will sound if door is opened and a resident wearing a transmitter is within 10-14 feet from door monitor. If the alert is triggered when a transmitter is too close or too far from door, or if you can pass thru door, call for technical assistance.