


Quick Start Instructions

Step 1

FLOOR MAT PLACEMENT


- Place the Floor Mat in desired location, such as a critical doorway or next to a bed.
- Place monitor out of reach of the resident. Suitable mounting locations include back of headboard, wall or under the bed so that the resident cannot tamper with monitor or reach the "Reset" button.

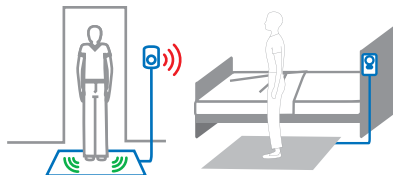

Step 2

PLUG IN FLOOR MAT


- Plug the Floor Mat into the bottom of the desired fall alarm monitor.
- Be sure to lace the cord through the strain relief by gently pinching the cable into a loop and lacing it over the "T-shaped" strain relief.

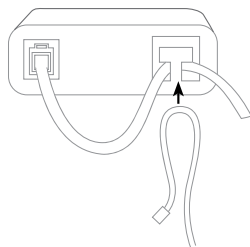

Step 3

ADJUST SETTINGS

- To adjust the volume, tone and delay, open the battery compartment in the back of the monitor to access the controls.
- Use a small screwdriver to adjust the volume to the desired level.
- Slide tone switch to the desired 3, 2 or 1 beeps.
- Delay comes set at 0 seconds. If caregiver deems it safe and necessary to have a delay, slide the switch to 1 or 2 seconds.

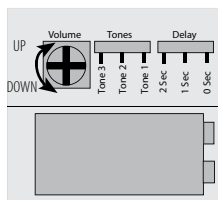

Step 4


TEST SYSTEM

- Step on the Floor Mat to set off the alarm.
- The monitor is now active. Alarm will sound whenever weight is on the Floor Mat.
- When the alarm is activated press the "Reset" button. This will silence the alarming mode.

Step 5

TAMPER PROOF MONITOR

To disarm the "Reset" button, slide switch on the side of the monitor to "Caregiver Key™ ON" (patented). This allows only the caregiver to turn off the monitor using the Caregiver Key™ (patented) by passing the key over the red circle on the monitor marked "Caregiver Key." A Caregiver Key™ (patented) is included with the Tamper-Proof™ monitor.

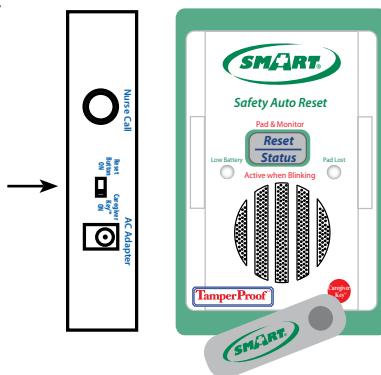

Floor Mat Fall Alarm Monitor

Warning: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. Test this device before each use. Read the instructions and Legal Disclaimer.

Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.

The Smart Caregiver Corporation Floor Mat Fall Alarm Monitor is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including, but not limited to:

- Direct patient supervision
- Adequate training for staff personnel for elopement
- Testing of the system before each use.

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

WARNING: The manufacturer does not claim that this device will stop elopement and or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.


Need Set-up Help?

Call Our Friendly Customer Service Reps at (800) 650-3637



1229 N. McDowell Blvd., Petaluma, CA. 94954
 sales@smartcaregivercorp.com
 www.smartcaregivercorp.com
 Toll Free: (800) 650-3637

Floor Mat Fall Alarm Monitor

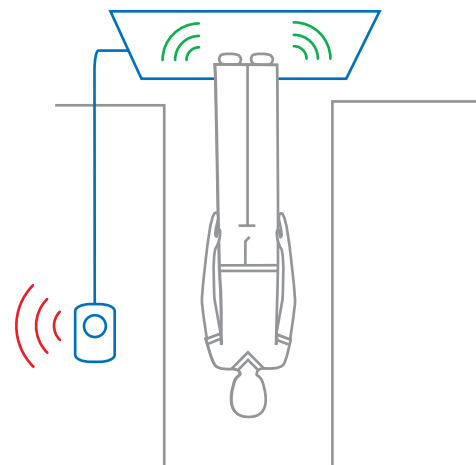

Installation Instructions

Installation Instructions

Floor Mat Fall Alarm Monitor



1229 N. McDowell Blvd., Petaluma, CA. 94954
sales@smartcaregivercorp.com
www.smartcaregivercorp.com
Toll Free: (800) 650-3637



Floor Mat Fall Alarm Monitor
A weight-sensing floor mat functions with any TL-2100 Series Fall Monitor to signal a fall alarm when weight is placed on the floor mat. The weight-sensing floor mat may be placed at the side of a bed or in a doorway to alert the caregiver when a resident arises or attempts to depart. This Floor Mat features a phone-style plug. Floor mats are made out of heavy-duty soft vinyl and are moisture-sealed for long lasting use. "Extra Long" Floor mats for bedside use are available.

Floor Mat Contents:
-Heavy-duty soft vinyl Floor Mat
-Durable and moisture-sealed
-Can be used with Nurse Call System

Monitors and Optional Accessories (sold separately):

- Economy Fall Alarm Monitor (TL-2100E)
- Safety Auto-Reset™ Fall Alarm Monitor (TL-2100S)
- Safety Auto-Reset™ w/Voice Fall Alarm Monitor (TL-2100V)
- Caregiver Key