



Quick Start Instructions: TL-3005SYS

1 Mount Door Monitor to Wall

A Peel off yellow backing from adhesives (3).

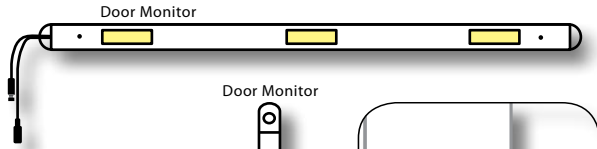


B Adhere door alarm vertically centered to the outside of door frame on the handle side of the door with cords facing up or down (in direction of power source).

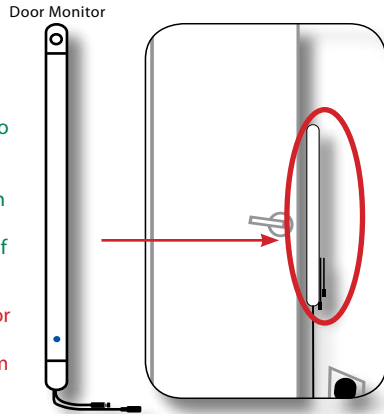

Warning: If mounting door monitor near metal frame, mount at least 8" away from metal frame.

2 Plug in AC Adapter

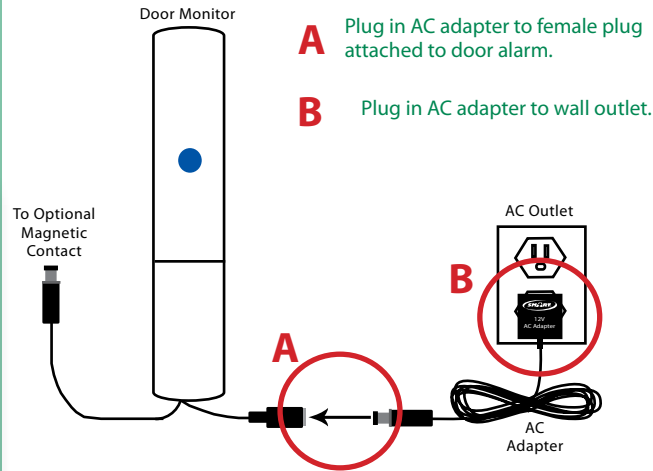

A Plug in AC adapter to female plug attached to door alarm.


B Plug in AC adapter to wall outlet.

Note: Alarm will illuminate and beep to indicate system is functioning.

3 Activate Resident Transmitter

A Place orange magnet on the white transmitter between 3 o'clock and 6 o'clock with the notched end of the wristband at 6 o'clock.

B Transmitter will blink 3 times to indicate transmitter is turned on.

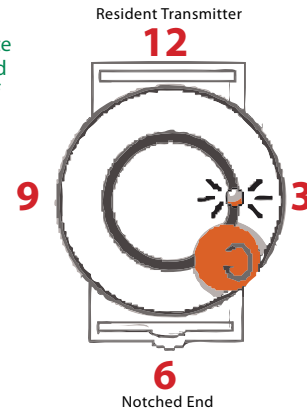

Warning: Transmitter bands are not to be placed on a wheelchair, walker or near jewelry. The correct placement is on a resident's wrist.

4 Test System

A Test the active range of the door monitor by approaching the door with a resident transmitter (turned on) from all different directions, making sure that alarm goes off from desired distances.


B To silence alarm, place the orange magnet over the blue sticker on the door monitor.

Orange Magnet



C To retest, wait approx. 20 seconds for monitor to blink (single light) before system is ready.

D Attach transmitter to resident's wrist with included strap.

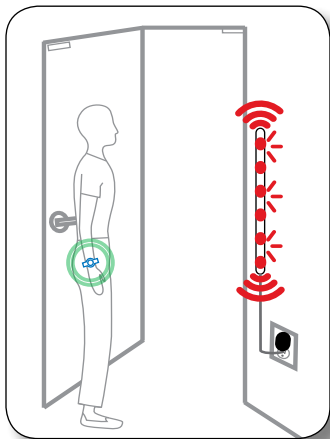


Warning: If door monitor goes off when transmitter is too close or too far from door or if you can pass thru door, call for technical assistance.



5 Mount Magnetic Contact Switch (Optional)

When using the magnetic switch, the alarm will only sound when door is opened and resident wearing a wristband is within the monitored field.


A Screw magnetic contact (see below) to underside of door frame with wording facing open door. Lead cord along outside of door frame.

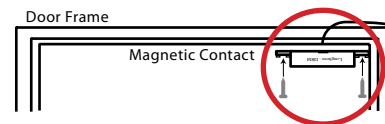


B Screw magnet (see below) to door with arrow facing up.

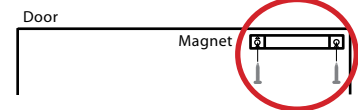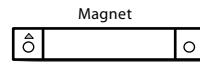

C Magnetic contact and magnet must align perfectly to function.

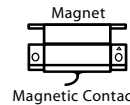

D Plug in magnetic contact switch to male plug attached to door alarm.

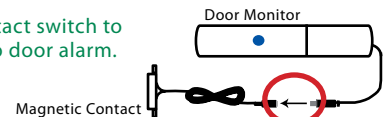

6 Testing System with Magnetic Contact Switch

A Test the door monitor with the magnetic contact switch installed by opening the door with an activated resident transmitter in hand. Door monitor will sound when door is opened.


B To silence alarm, place the orange magnet over the blue sticker on the door monitor.

Orange Magnet



C To retest, wait approx. 20 seconds for monitor to blink (single light) before system is ready.

D Attach transmitter to resident's wrist with included strap.

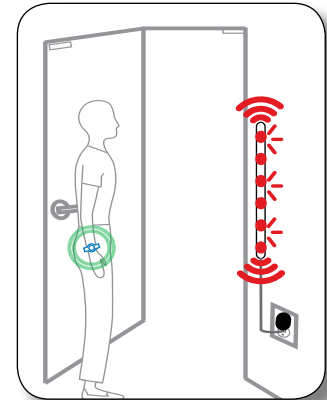

Warning: Door monitor will sound if door is opened and a resident wearing a transmitter is within 20 feet from door monitor. If door monitor goes off when transmitter is too close or too far from door or if you can pass thru door, call for technical assistance.

To meet industry standards, door bars & wristbands should be tested at regular intervals to assure proper function. A standard procedure should be initiated at the facility for this testing. It is recommended that a tester be purchased for all facilities using SMART Anti-Wandering Door Systems.





Need Set-up Help? (800) 650-3637

- Wired Alarm Light (TL-2019)
 - Wireless Light (TL-2018)
 - Magnetic Door Lock (TL-2025)
 - Central Monitoring Unit (TL-2015REV-01)
 - Door Alarm Reset/Tester & Wristband Tester (DT-01)
 - Wireless Key Pad (TL-2010WKP)
 - Wall Mount Reset Button (TL-2014W)
- TL-3005 SYS Optional Accessories (sold separately):**


- 2 Double Sided Tape Strips
- 1 AC Power Adapter (AC-03)
- 1 Caregiver Key (CK-01D)
- 1 Resident Wristband Transmitter (TL-2012S)
- 1 Door Alarm Bar (TL-3005)

TL-3005SYS Contents:

About The DepartAlert™ Anti-Wandering System:

The DepartAlert™ anti-wandering system projects a sensor field and alarms visually and audibly when a resident wearing a wristband transmitter passes within the sensor field's range. Door monitor can be configured for use with a Central Monitoring Unit that will identify specific resident and doorway for up to 999 residents. DepartAlert™ can also be configured for double doors.

TL-3005SYS DepartAlert™ Anti-Wandering System

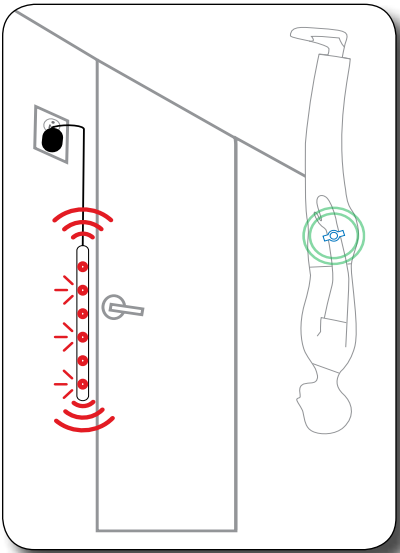

TL-3005SYS Door Watch System™



Smart Caregiver Corporation
sales@smartcaregivercorp.com
Toll Free: 800-650-3637

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

The Smart Caregiver Corporation Fallguard™ monitors are designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including, but not limited to:

- a.) Direct patient supervision b.) Adequate training for staff personnel for elopement c.) Testing of the system before
- claim that this device will stop elopement and/or stop falls. This device is to help augment caregivers comprehensive resident mobility management program. **Test this device before each use. Read the instructions and Legal Disclaimer.**
- Warnings: Failure to comply with warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by caregiver. The manufacturer does not warrant that this device will stop elopement and/or stop falls. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, the goods purchased are being sold on a "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.
- The Smart Caregiver Corporation Fallguard™ monitors are designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good care giving practices including, but not limited to:

Important Information:

DepartAlert™ anti-wandering system relies on wireless technology which is subject to physical and environmental considerations. These products do not have an out of range function and as such should be tested periodically in the setting in which they are to be used to understand their area of effective operation. Please be aware that the transmitter will not be 100% accurate if it is out of range at anytime. It is the end-user's responsibility to make sure that the product is used correctly and within range of each other. Wireless systems and monitors are intended as an adjunct to good caregiving practices and are not a substitute for proper staffing and patient management practices. We recommend that all personnel receive periodic training in the operation of these systems and that the systems are tested before each use. It is important to read and follow the directions.

TL-3005SYS DepartAlert™ Anti-Wandering System

Installation Instructions

TL-3005SYS DepartAlert™ Anti-Wandering System

